Clario *Toni*

SUCTION PUMP

- EN Patient Instructions for use
- DE Gebrauchsanweisung für Patienten
- FR Mode d'emploi
- IT Istruzioni per l'uso per i pazienti
- NL Gebruiksaanwijzing voor patiënten
- SV Patientbruksanvisning
- FI Käyttöohjeet potilaalle
- NO Pasientbruksanvisning
- ES Instrucciones de uso para el paciente
- PT Instruções de utilização para o paciente

Assistance

Read the entire instruction manual before trying to operate the device.

If you have any questions about your Clario Toni suction pump, please contact your healthcare provider/physician at the number below:

HEALTHCARE PROVIDER/PHYSICIAN CONTACT INFORMATION:

Keep this Patient Instructions for use booklet in an easily accessible location.

Table of contents

Warnings and safety instructions.....	4
Description of the device.....	9
Set up instructions.....	13
Preparation for use.....	14
Operating instructions.....	18
Emptying the canister.....	22
Replacing the valve discs.....	23
Replacing the rechargeable battery.....	24
Battery data and test.....	25
Battery calibration.....	27
Troubleshooting.....	28
Cleaning guidelines.....	30
Signs and symbols.....	32
Warranty and servicing.....	34
Disposal.....	35
Technical specifications.....	36

Warnings and safety instructions

WARNINGS

The individual responsible for operating this device on a daily basis (patient, caregiver) must be able to read, understand and follow directions provided in this instructions for use and by the healthcare professional. The individual must be able to hear and respond to auditory signal.

Only use the Clario Toni suction pump on the person for whom it was ordered and only for its intended use.

Do not perform the therapy without your healthcare provider's supervision.

The Clario Toni is approved exclusively for the use as described in these Patient Instructions for use. Medela can only warranty the safe functioning of the system when the Clario Toni is used in combination with the original Medela accessories.

Please read and observe these warning and safety instructions before operation. These Patient Instructions for use must be kept with the device for later reference.

Please note that these instructions for use are a general guide for the use of the product. Medical matters must be addressed by a physician.

- If you are dependent on the device for airway suctioning and a breakdown can lead to a critical situation, you must have a replacement device at hand.
- The connecting tubing delivered with the device must always be connected to a sterile catheter as prescribed by your physician.
- Do not modify the equipment.
- Clario Toni is EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and IEC 60601-1-2:2014 4th Edition according to clause 7 and 8.9. Clario Toni is suitable for use in home and clinical environments.
- Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the Clario Toni and should be kept at least a distance 1ft (30 cm) away from the equipment.

General safety at home

- Clario Toni should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, Clario Toni should be observed to verify normal operation in the configuration in which it will be used.
- Keep the power supply cord away from hot surfaces.
- Keep the mains plug and the on/off switch away from moisture.
- Never pull the mains plug out of the fixed mains socket by pulling on the power supply cord!
- Never leave the device unattended when it is switched on.
- The pump must stand upright during use.
- Never use the device at high room temperatures, while bathing or showering or if you are very tired.
- Do not use extension cords with Clario Toni pump.
- Keep the power supply cord where you will not fall or trip over it.
- Never place the power supply cord around your neck.
- Keep the Clario Toni pump clean and dry.
- Never place the pump in water or liquids.

- If the pump gets wet, rub with dry towel. Do not dry in a microwave.
- Keep the pump away from children and pets.
- Keep the pump protected from direct sunlight.
- Prevent the pump from falling to the floor during sleep.
- Do not touch the pump when it has fallen into water. Unplug device immediately from electrical outlet.

When you leave the house

- Make sure the LED indicator is not blinking when you leave the house.
- For your convenience put the pump into the optional carrying bag and store the tubing safely in the tubing bag.
- Don't forget to take a spare jar/tubing and the power supply with you.

These instructions for use must be kept for later reference.

Description of the device

Introduction

Your physician has prescribed the Clario Toni for airway suctioning. This suction pump from Medela is a high-quality suction pump that combines easy handling and cleaning with safety features to ensure optimal operation.

Intended use

The Clario Toni suction pumps are indicated for aspiration and removal of secretions, bodily fluids and foreign objects from a patient's airway or respiratory support system in the nasal, pharyngeal and tracheal areas of pediatric and adult patients. The suction pump is suitable for use at home, in the hospital, in the car or outdoors.

Indications

The Clario Toni suction pump is indicated to clear the airways either through the nose, mouth or tracheostomy tube.

Contraindications

The device is not suitable for use for prolonged suctioning.

Intended user

The Clario Toni should only be operated by properly instructed users.

Expected service life

The expected service life of the device is seven years.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Subject to change.

Versions of the pump

Clario Toni AC/DC

Front of the pump

LED

Vacuum regulator

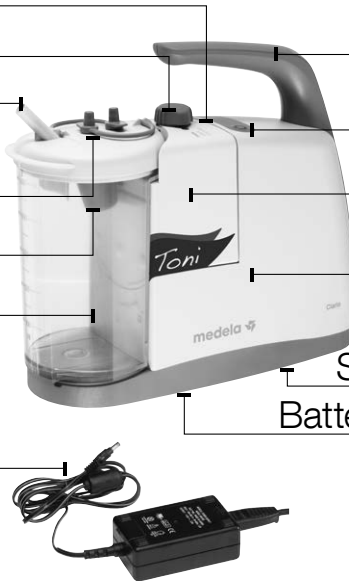
Suction port

Plugs for closing

Floater

Canister

Power cord

Handle

On/off button

SafetyChamber

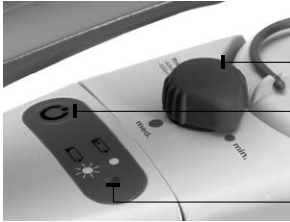
Motor unit

Mains socket

Specification plate

Battery compartment

Operating elements and battery status

Vacuum regulator

On/off button

Rechargeable battery LED

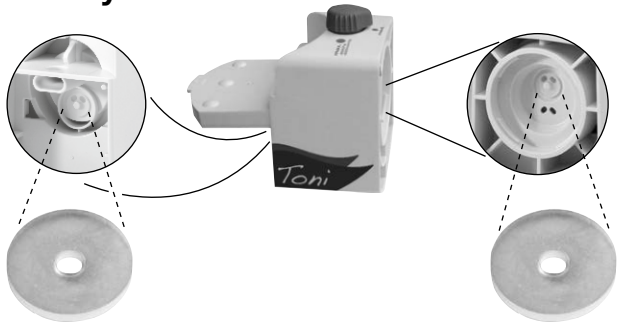
not lit Battery is fully charged

lit up – Battery is being charged
 – Pump is plugged in

blinks slowly Battery is soon empty

blinks rapidly Battery is defective

Valve discs in the SafetyChamber

Position of the membrane

Set up instructions

Check initial delivery

Check the delivery package of the Clario Toni for completeness and general condition.

1 x Clario Toni suction pump

1 x Disposable fingertip tubing

1 x Power adapter

1 x Instructions for use

WARNING

The connecting tubing supplied with the device must never come into direct contact with the suction area. A sterile suction catheter must always be used (risk of infection).

Accessories

Please ask your healthcare provider / physician in case of missing parts or for additional accessories.

Preparation for use

WARNINGS

For use only by persons who have been adequately trained in the use of suction pumps for airway suctioning by their healthcare provider/physician.

CAUTIONS

- The Clario Toni must remain in an upright position during use.
- Sterile accessories must be checked on the integrity of the packaging before use.
- Non sterile and reusable accessories must be cleaned and disinfected before use.

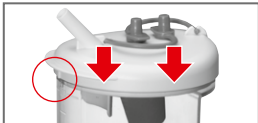
Checks before use

- **Check the Clario Toni system before use for damage of the power cord or plug, obvious device damage or safety defects and proper functioning of the device.**
- **Make sure that the rechargeable battery is calibrated according to chapter "Battery calibration".**
- **Check all accessories prior to use:**
 - **Canister and lid for cracks, brittle and flawed spots. Replace if necessary.**
 - **Tubing for cracks, brittle areas and that connectors are firmly attached. Replace if necessary.**

Assembly of the canister

1.  Attach the floater to the canister.

2.  Attach the lid to the canister. Mind the markings on the front of the canister and the lid.

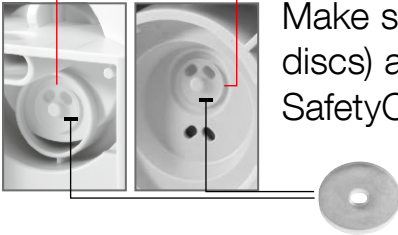
Assembly of the SafetyChamber

1. Open the lid of the SafetyChamber.

2. Make sure that both valves (silicone discs) are attached to the SafetyChamber.

3. Make sure that the lid of the SafetyChamber is closed well.

4. a) Hold the SafetyChamber and tilt 45° to the left to attach it to the motor unit.
b) Turn the SafetyChamber to an upright position until you hear a click.

Attaching the canister to the pump

1.  Attach the canister to the pump. Press firmly.

2.  Attach the disposable fingertip tubing to the suction port.

CAUTION

Do not use tubing or other sterile accessories if the sterile packaging is damaged. Do not reuse single use or sterile products.

Operating instructions

CAUTIONS

The Clario Toni is to be set up in such a way, that a separation from the mains supply can be easily managed. Calibrate the rechargeable battery before the first use.

Connect Clario Toni to mains power

1. Check the pump before use following the instruction in chapter "Preparation for use".

2. **All versions:** Plug in the mains plug of the power cord to a fixed mains socket.

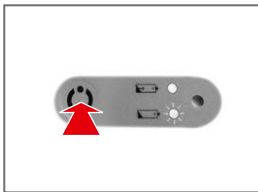
or


If the pump is operated using the optional car connection cable: Connect the optional 12V DC connection cable to the socket on the side of the pump and connect it to the 12V DC power source (cigarette lighter in the car).


or


If the pump is operated with the rechargeable battery: No connection to a power source is needed. Make sure that the battery was charged for five hours.

Functional check with water

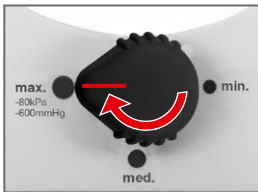1.  Press the On/off button to switch on the pump.

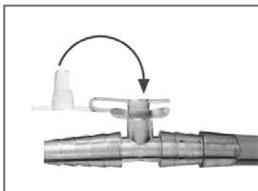2.  Set vacuum regulator to position "max.".

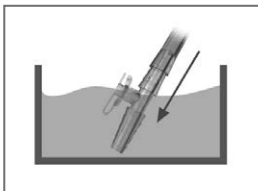3. 
 - a) Attach the fingertip tubing.
 - b) Plug fingertip port.

4.  Place fingertip into water and check if there is suction.

5. If there is suction, proceed with step 6. If not, see chapter "Troubleshooting".

6. Use catheter and vacuum setting as prescribed by a physician.

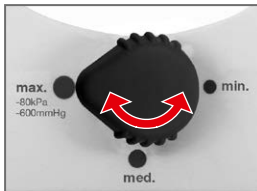
WARNING

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.


Changing vacuum level

1.

Set vacuum regulator to the necessary position.

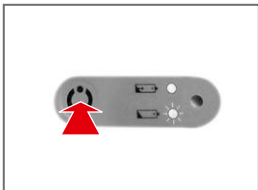Placing out of operation after use

1. 

Press the On/off button to switch off the pump.
2. **If the pump is connected to a fixed mains socket:**

Disconnect the mains plug from the fixed mains socket.

or

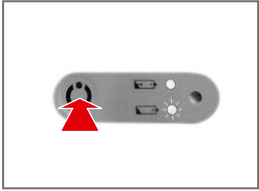
If the pump is connected to a 12V DC power source:

Disconnect the 12V DC plug from the 12V DC power source.
3. Clean and disinfect the Clario Toni.
See chapter "Cleaning guidelines".

Emptying the canister

Safety related tip

On a daily basis or latest when the fluid level reaches 550 ml and the floater stops suction into the canister the canister should be emptied.

1.  Press the On/off button to switch off the pump.

2.  a) Disconnect the disposable fingertip tubing from the suction port.
b) Remove canister from the pump.

3.  For transportation always seal the canister with the plugs.

4. Empty canister and dispose of fingertip tubing in accordance with local guidelines and inhouse guidelines.

5.  a) Attach a clean canister.
b) Attach a new disposable fingertip tubing to the suction port.

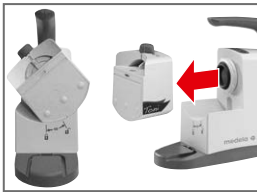
6. Switch on the pump and set the vacuum as prescribed by a physician.

Replacing the valve discs

1. Remove the canister.


2. a) Turn SafetyChamber 45° to the left.
b) Remove SafetyChamber by pulling it off the pump.

3. Open the SafetyChamber.

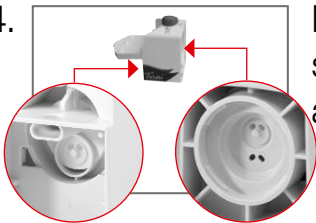4. Replace defective valve discs. Make sure that the new valve discs are attached correctly.


5. Attach the SafetyChamber to the pump as described in chapter "Preparation for use".

Replacing the rechargeable battery

WARNINGS

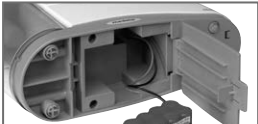Before replacing the rechargeable battery, switch off the Clario Toni and pull the mains plug from Clario Toni out of the fixed mains socket.

1.  Open the lid of the battery compartment on the bottom of the Clario Toni.

2.  Detach the battery cable by pulling the plug.

3. Remove the old rechargeable battery.

4. Insert the new rechargeable battery.

5.  Reattach the battery cable by plugging in the plug.

6. Close the lid of the battery compartment. Make sure that the lid is closed well and that the battery cable is not clamped or pinched.

7. Calibrate and test the rechargeable battery (see chapters Battery data and test/Battery calibration).

Battery data and test

⚠ CAUTIONS

Only charge the rechargeable battery when the charging indicator blinks slowly.

Duration of pumping

If the battery is fully charged, the operating time at maximum vacuum is approximately > 50 minutes.

Charging time (charged up to 100%)

Approx. 5 hours

Battery status

not lit Battery is fully charged

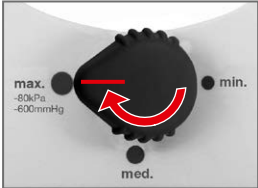
lit up – Battery is being charged
– Pump is plugged in


blinks slowly Battery is soon empty

blinks rapidly Battery is defective

Testing the rechargeable battery

1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed mains socket.

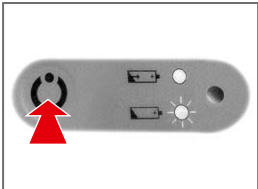3.  Set vacuum regulator to position "max.".
The image shows a black, circular vacuum regulator knob with three positions: "max." (top left), "min." (top right), and "med." (bottom). A red arrow points from the "min." position to the "max." position, indicating the adjustment.

4.  Switch on the Clario Toni.
The pump will start running.
The image shows a grey control panel with a power button on the left, a battery icon, a light icon, and a small circular indicator. A red arrow points to the power button.

5. Measure the time until the pump comes to a stop.
The battery is ok, when the running time is > 50 minutes. If the running time is < 50 minutes, repeat test or calibrate the battery (see next page).

Battery calibration

⚠ CAUTIONS

Calibrate the rechargeable battery before first use and after storage duration > 2 months. Replace battery if the calibration is not successful.

Calibrating NiMH rechargeable battery

1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed mains socket.

3.  Switch on the Clario Toni.
The pump will start running.

4. Let the pump run until the rechargeable battery is empty and the pump comes to a stop.
5. Connect the Clario Toni to a fixed mains socket.
The rechargeable battery is being charged.
6. Wait until the charging indicator goes out.
7. Repeat step 1 to 6. The Clario Toni AC/DC is then ready to use.

Storage of the rechargeable battery

In order to keep the self-discharge at a minimum, store the Clario Toni and the removed replacement rechargeable batteries at temperatures below 25 °C (77 °F). Repeat the calibration process every 60–90 days.

Troubleshooting

If the motor is not running

Check if:

- the Clario Toni is switched on.
- the mains voltage is correct and if the mains plug is inserted correctly into the wall socket.
- the internal battery is charged (only under battery operation).

If the pump is too loud

Check the position and seal of the valve discs inside the SafetyChamber. If necessary reposition according to chapter "Replacing the valve discs".

If the pump does not suction strongly enough

Check if:

- the vacuum regulator is set to the correct and prescribed suction level.
- the tubings are not defective or broken. If necessary, replace.
- all plug-in connections are tight.
- the suction canister/lid have no cracks, brittle areas, discolouration. If necessary, replace.
- the seal of the valve discs inside the SafetyChamber is tight.
- the membrane is clean and has no tears.

Running time from battery insufficient

Calibrate the rechargeable battery (see chapter "Battery calibration").

No LED lit

The rechargeable battery is fully charged.

LED lit up

- The rechargeable battery is being charged.
- The pump is connected to the mains.

LED blinks slowly

The rechargeable battery is soon empty. The battery power is sufficient for about 10 minutes of pumping.

LED blinks rapidly

The rechargeable battery is defective, please contact your healthcare provider.

CAUTIONS

If the fault cannot be rectified, please contact your healthcare provider.

Cleaning guidelines

WARNINGS

Before cleaning the device, pull the mains plug out of the wall socket.

General notes

- Follow the cleaning instructions given by your healthcare provider.
- Wear protective gloves for cleaning / disinfection.
- Dispose of fluids such as blood and secretions and the parts contaminated with them according to your healthcare provider's instructions.

Water

Use only the purest quality of water for cleaning. Water hardness is a serious consideration since deposits left on medical products may not be properly decontaminated. Use deionised water in order to reduce this problem.

Disposable products

These are single use products not intended to be reused. Reuse could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.

Disassembly

Separate all individual parts before cleaning and disinfecting.

Cleaning canister and lid

Clean components in hot water (60–70 °C) containing a detergent with a pH range between 6.0 and 8.0 only, in order to avoid damaging the instruments and containment devices.

1. Soak all parts thoroughly with warm, soapy water (60–70 °C) or in enzymatic detergent for 1–5 minutes.
2. Remove visible dirt with a cleaning tool – general purpose cleaning brushes, such as pipe cleaners or non-abrasive lint cloths. Rinse thoroughly in clear water. Allow to dry.
3. Check the parts for visible dirt and repeat these steps if necessary. Alternatively the canister/lid can be placed in a dishwasher for cleaning.

Cleaning pump housing, mains cable and plastic accessories

Wipe with detergent recommended by your healthcare provider. Use clean cloth to dry.

Signs and symbols

This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the class of the pump.

This symbol indicates a class II device.

IP21

This symbol indicates the protection against harmful effects due to ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates that the device should not be used after the end of the year and month shown.

This symbol indicates a single use device. Do not reuse the device.

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

REF

This symbol indicates manufacturer's catalogue number.

SN

This symbol indicates manufacturer's serial number.

This symbol indicates the legal specifications of the pump.

This Symbol indicates a type BF applied part.

This symbol indicates MR UNSAFE.

LOT

This symbol indicates manufacturer's batch code.

This symbol indicates to follow instructions for use.

This symbol indicates that interferences may occur in the vicinity of equipment marked with this symbol.

STERILE

This symbol indicates the device is sterilized using ethylene oxide.

This symbol indicates to consult instructions for use.

This symbol indicates the manufacturer.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates a CAUTION or WARNING associated with the device.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates safety related tip.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

pcs This symbol indicates number of items.

This symbol indicates do not use the device if package is damaged.

This symbol indicates the maximum vacuum level of the pump.

This symbol indicates the flow levels of the pump.

This symbol indicates the number of items n that the content is sufficient for.

This symbol indicates the electrical specifications of the pump.

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates alternating current.

This symbol indicates direct current.

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates the weight of the pump.

This symbol indicates a carton package.

This symbol indicates the dimensions (h x w x d) of the pump.

This symbol indicates to keep the device away from sunlight.

This symbol indicates a Prescription Device. CAUTION: U.S. Federal law restricts this device to sale by or on the order physician (for US only).

This symbol indicates to handle the fragile device with care.

Warranty and servicing

Warranty

Warranty period is for 2 years (6 months for the rechargeable battery) after date of delivery. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons.

Servicing

The Clario Toni pump should not require maintenance. If a Clario Toni pump fails within the warranty period due to a manufacturing defect, it will be replaced. The original pump will need to be returned to the supplier.

Disposal

Ask your healthcare provider how to dispose of used tubings.

Technical specifications

CAUTIONS

Do not operate the Clario Toni pump above 9,842 ft above sea level (3,000 m above sea level).

Transport/Storage conditions

The Clario Toni pump and accessories must remain in the packaging for storage and stored at a temperature range from -4°F to $+122^{\circ}\text{F}$ (-20°C to $+50^{\circ}\text{C}$).

Rechargeable battery $+41^{\circ}\text{F}$ to $+77^{\circ}\text{F}$ ($+5^{\circ}\text{C}$ to $+25^{\circ}\text{C}$)

Operating conditions

The Clario Toni pump and accessories must be operated within a temperature range of $+41^{\circ}\text{F}$ and $+104^{\circ}\text{F}$ ($+5^{\circ}\text{C}$ and $+40^{\circ}\text{C}$). Do not operate the products in extreme cold or heat. Avoid high humidity (15–93% is permissible).

Protection class IP21

The Clario Toni pump is protected against ingress of dripping water.

min. vacuum:

– 70 mmHg / – 9 kPa (+/– 20 mmHg / +/- 2.6 kPa)

med. vacuum:

– 115 mmHg / – 15 kPa (+/– 30 mmHg / +/- 4 kPa)

max. vacuum:

– 600 mmHg / – 80 kPa (– 90 mmHg / – 12 kPa)

Calculated at 0 meter, atmospheric pressure:
1013 hPa. Please note: vacuum levels may vary
depending on location (meters above sea level,
atmospheric pressure and temperature).

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Medela AG
Medela UK Ltd.
Medela LLC